




## 1 Series

Toggle to show/hide panel of series and related studies.

## 2 Settings

Customize toolbar items, keyboard shortcuts, hanging protocols and other viewer settings.

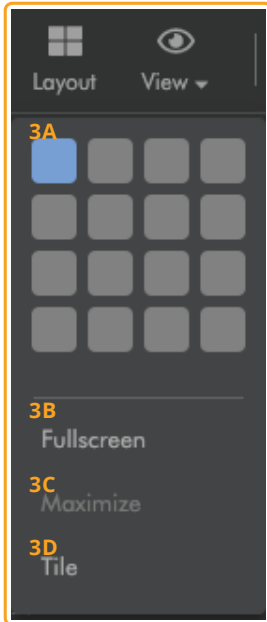


## 3 Layout

- 3A From the grid, select the number of view panels to display simultaneously.
- 3B Make browser window full screen.
- 3C Display selected image in a single viewport.
- 3D Display images from a single series in all viewports.

## 4 View

- 4A Toggle to show/hide annotations.
- 4B Toggle to show/hide GSPS annotations.
- 4C Toggle to show/hide patient information.
- 4D Toggle to show/hide overlays.
- 4E Toggle to show/hide reference lines.
- 4F Toggle to show/hide ruler.
- 4G Hide all study information.
- 4H Show all study information.

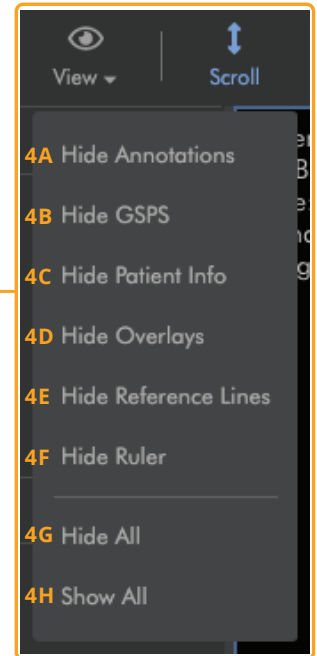

## 5 Scroll

Scroll through images in a series. Use mouse wheel or up/down keys to scroll.

## 6 Zoom

Zoom in and out of image.

## 7 Move

Move image by dragging image to anywhere on screen.

## 8 Rotate

Rotate image in 90 degree increments clockwise.

## 9 Levels

Adjust the window width and window center for the selected series. *To change mouse controls for window width and window level, open Settings, scroll to the bottom of the General Settings menu, and select from Window Level settings.*

## 10 Presets

View and apply window/level presets. *To add or edit presets open Settings and click the Window Level menu.*

## 11 Palettes

Apply a preset or custom color palette to the displayed study. Default options include default (grayscale), invert, hot iron, hot blue, and fire.

## 12 Length

Measure a length between two points on an image. Measurements display in millimeters to 100, then in centimeters.



## 13 Angle

Measure an angle. Select three points using the left mouse button. The angle at the middle point will be measured and displayed.

## 14 Rectangle

Create a rectangle annotation and calculate the dimensions. Rectangles are also used with PET CT studies to generate SUV measurements.

## 15 Label

Draw an ellipse around an area and label it. Dimensions are not included by design. Labels can be applied to an existing annotation by clicking "edit label" in the right click/context menu.

## 16 Text

Create a text annotation. Edit the size of an annotation by clicking inside the text and dragging the sizing box.

## 17 Color

Select the color you want an annotation to be. If you have multiple annotations with different colors, only measurements for annotations with the same color will show at the same time. You can toggle between different colors as needed.

## 18 Play

View images as a movie.



Please note that the ProViewer toolbar may vary slightly depending on user viewer settings. For assistance with configuration please visit [support.ambrahealth.com](http://support.ambrahealth.com).

